

Alexis® O


Wound Protector-Retractor

SETUP GUIDE


Prepare the surgical site according to standard procedure, making sure the skin is clean and dry.



- 1** Make an incision along the marked incision line.

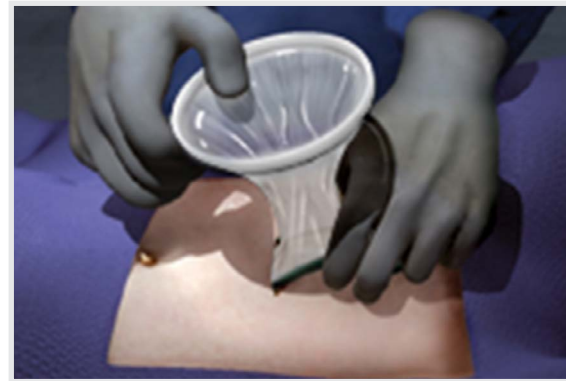


- 2** Insert the Alexis O wound protector-retractor's green ring into the operative site.

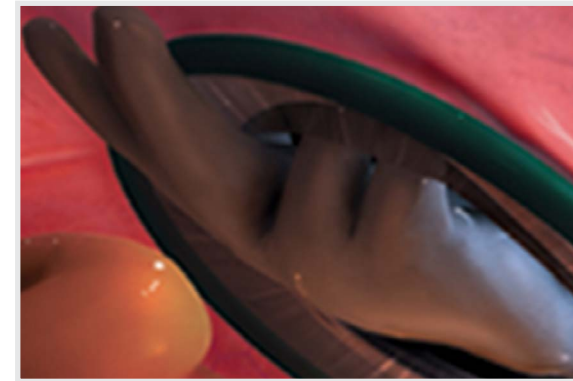

- 3** Carefully check to ensure that no bowel or tissue entrapment has occurred before retraction is initiated.




- 4** Gently grasp the white retraction ring at 10 and 2 o'clock and pull up until the green ring sits tightly against the peritoneal layer. Flip the outer ring **inward or outward** until the desired retraction is achieved.

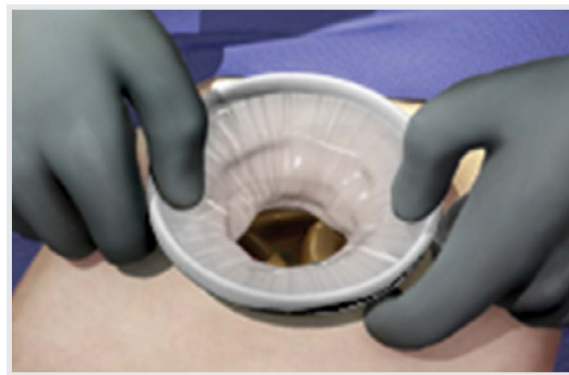

- 5** Double-check to ensure that no bowel or tissue entrapment has occurred. (See Figure 3.)



- 6** Perform the procedure through the retracted and protected incision site.

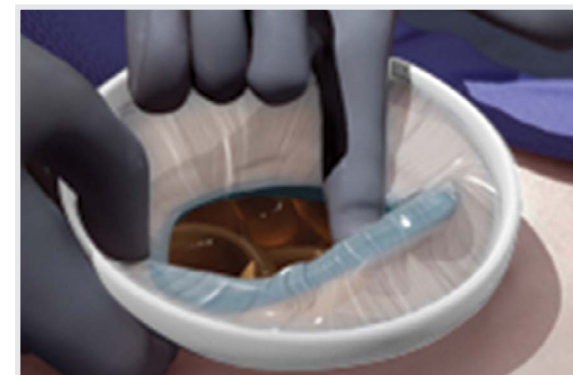

- 7** Retrieve the Alexis O wound protector-retractor by simply removing the green inner ring from the peritoneal cavity.

Visit [appliedmedical.com/alexis](https://www.appliedmedical.com/alexis) for more information.

This information is not intended to replace the Instructions for Use (IFU). Please refer to the IFU for the indications, contraindications, warnings, precautions, instructions and other information.

Devices listed may not be approved in all markets.

Please contact your Field Implementation team member for more information on availability.

© 2021 Applied Medical Resources Corporation. All rights reserved. Applied Medical, the Applied Medical logo design, and marks designated with the symbol ® are trademarks of Applied Medical Resources Corporation, registered in one or more of the following countries: Australia, Canada, Japan, South Korea, the United States, the United Kingdom, and/or the European Union. 224721-EN-USA-C