

Alexis® O


Wound Protector-Retractor

SETUP GUIDE


Prepare the surgical site according to standard procedure, making sure the skin is clean and dry.




- 1 Make an incision along the marked incision line.

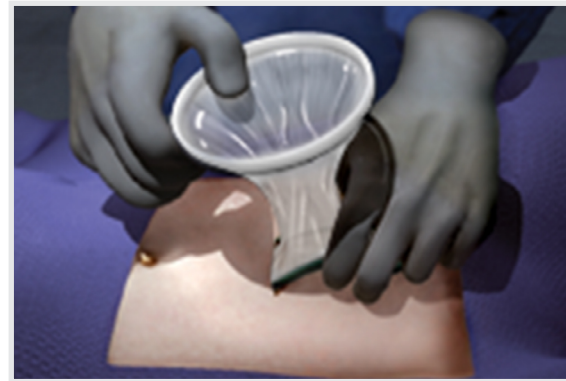


- 2 Insert the Alexis O wound protector-retractor's green ring into the operative site.

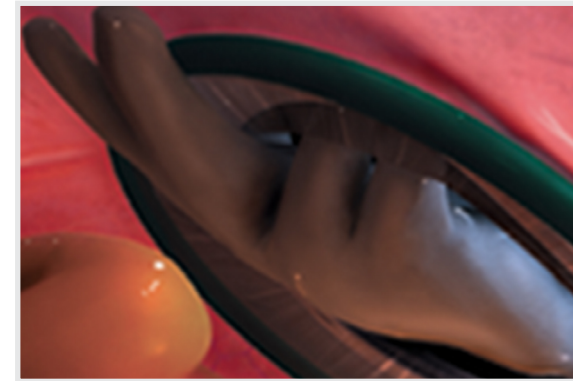

- 3 Carefully check to ensure that no bowel or tissue entrapment has occurred before retraction is initiated.

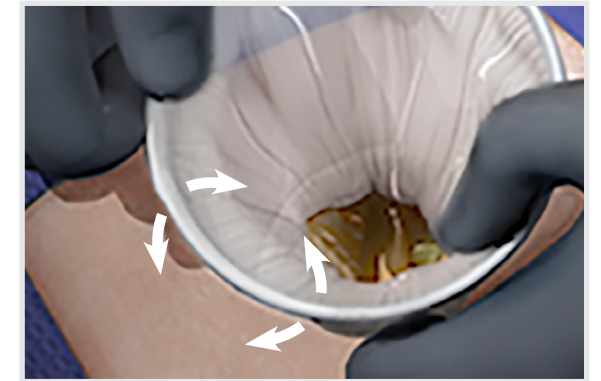


- 4 Gently grasp the white retraction ring at 10 and 2 o'clock and pull up until the green ring sits tightly against the peritoneal layer. Flip the outer ring **inward or outward** until the desired retraction is achieved.

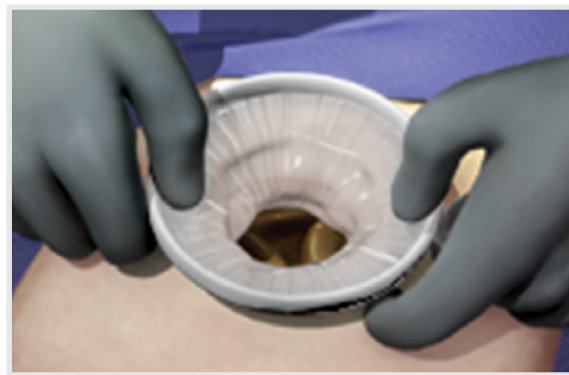

- 5 Double-check to ensure that no bowel or tissue entrapment has occurred. (See Figure 3.)



- 6 Perform the procedure through the retracted and protected incision site.

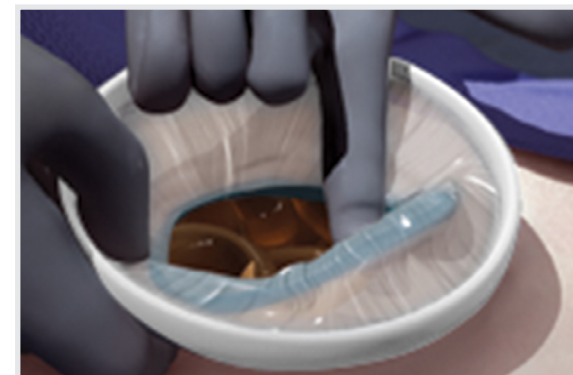

- 7 Retrieve the Alexis O wound protector-retractor by simply removing the green inner ring from the peritoneal cavity.

Visit [appliedmedical.com/alexis](https://www.appliedmedical.com/alexis) for more information.

Please contact your Applied Medical representative for more information on availability. This information is intended for dissemination exclusively to healthcare professionals and is not intended to replace labeling and Instructions for Use (IFU). Please refer to the IFU for the indications, contraindications, warnings, precautions, instructions and other information.

© 2021 Applied Medical Resources Corporation, Rancho Santa Margarita, CA. All rights reserved. Applied Medical, the Applied Medical logo design, and marks designated with the symbol ® are trademarks of Applied Medical Resources Corporation, registered in one or more of the following countries: Australia, Canada, Japan, South Korea, the United States, the United Kingdom, and/or the European Union. 02/2022 426622-EN-EUR-A