

Kii™ Dissecting Balloon Access Systems

SETUP GUIDE



1 Insert the introducer until the introducer and seal are locked into place.



2 Insert the dissecting balloon into the incision site with the inflation port perpendicular to the abdomen.



3 Remove the introducer.




4 Insert a 10mm endoscope.



5 Connect the inflation bulb, and twist the Luer lock connector clockwise onto the stopcock until firmly attached.



6 Inflate the dissecting balloon under visualization by compressing the inflation bulb.

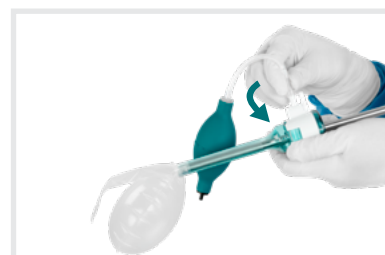

7 After completing dissection, untwist the Luer lock connector counterclockwise, and disconnect the inflation bulb.



8 Remove the endoscope for complete balloon deflation.

Kii dissecting balloon access systems are used in laparoscopic surgery for tissue separation of the extraperitoneal space.

Devices listed must be administered by a healthcare professional. Please contact your Applied Medical representative for more information on availability. This information is intended for dissemination exclusively to healthcare professionals and is not intended to replace labeling and Instructions for Use (IFU). Please refer to the IFU for the indications, contraindications, warnings, precautions, instructions and other information.

© 2026 Applied Medical Resources Corporation, Rancho Santa Margarita, California. All rights reserved.

Applied Medical, the Applied Medical logo design, and marks designated with the symbol ™ are trademarks of Applied Medical Resources Corporation, registered in one or more of the following countries: Australia, Canada, Japan, South Korea, the United States, the United Kingdom, and/or the European Union.

12/2025

569971-EN-B