
## CART

Instructions For Use  
Instrucciones de uso  
Mode d'emploi  
Istruzioni per l'uso  
Gebruiksaanweisung  
Instructies voor gebruik  
Bruksanvisning  
Bruksanvisning  
Käyttöohjeet  
Bruksanvisning  
Instruções de utilização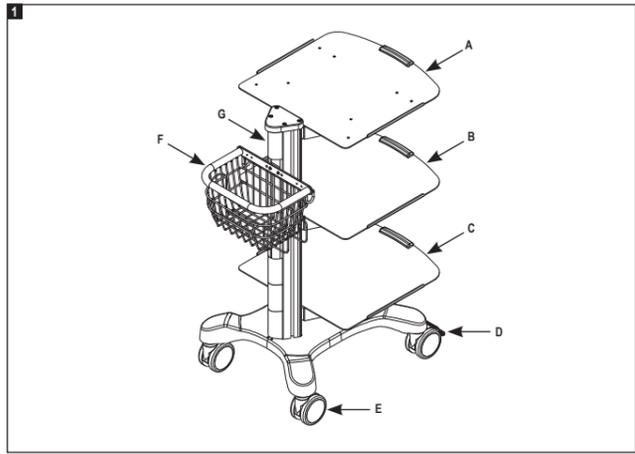

## ENGLISH

**INTENDED USE:**  
The Voyant cart (model EX150) is intended for use with Voyant® electrological generators.  
**DESCRIPTION:**  
The EX150 Voyant cart is designed for the support and transport of Voyant generators.

### KEY – ILLUSTRATIONS AND TERMINOLOGY:

A. Platform	E. Wheel
B. Middle Shelf	F. Basket
C. Bottom Shelf	G. Post
D. Brake Pedal	

**WARNINGS:**  

- Do not use if the product, protective packaging, or sterile packaging is damaged, opened unintentionally before use, and/or exposed to environmental conditions outside of those specified in the IFU.

**PRECAUTIONS:**  

- Prior to attaching the generator to the cart, lock the wheels to ensure maximum stability.
- Overloading greater than maximum load capacity may result in cart instability or damage.

### INSTRUCTIONS FOR USE:

#### • LOCKING AND UNLOCKING THE WHEELS

- Locking the Wheels
  - Place cart in desired location/position.
  - Step (once) on the brake pedal to instantly lock the caster holding the cart firmly in place.
  - Note: To ensure maximum cart stability/security, lock all brake pedals by repeating step 2.
- Unlocking the Wheels
  - Lift the brake pedal for each locked wheel.

#### • ATTACHING THE GENERATOR TO THE CART

- Prior to placing the generator on the cart, unscrew the rubber feet on the generator.
  - Place the generator on the center of the top platform of the cart.
  - To secure the generator on the cart, screw the rubber feet through the holes on top of the cart from underneath the platform.

Note: For detachment, follow steps 1-3 in reverse.

### CLEANING:

- Turn off the generator and unplug the power cable from the wall.
- Mild cleaning solutions or disinfectants, such as 70% isopropyl alcohol should be used. Thoroughly wipe all surfaces of the cart, following the cleaning procedures approved by your institution.

Note: To clean the top surface of the platform, remove the generator.

### HOW SUPPLIED:

The EX150 cart is supplied non-sterile.

### STORAGE AND HANDLING:

Handle with care. Product should be stored away from chemical fumes. Store between -15°C and 60°C (5°F and 140°F) and 20% and 95% relative humidity.

### REPORTABLE INCIDENTS:

If a serious incident occurs in relation to the device, notify Applied Medical and the local authority, if applicable or required within the region.

### SPECIFICATIONS:

General	EX150 Cart
Cart Weight	66.58 lb (30,20 kg)
Cart Depth	29.97 in (76,12 cm)
Cart Width	19.29 in (49,00 cm)
Cart Height	39.85 in (101,22 cm)
Maximum Loaded Cart Weight	253.97 lb (115,2 kg)
Ambient Temperature Range (Operating)	10°C to 40°C (50°F to 104°F)
Relative Humidity Range (Operating)	20% to 85% non-condensing
Atmospheric Pressure Range (Operating)	400 to 1060 mBar
Ambient Temperature Range (Storage)	-15°C to 60°C (5°F to 140°F)
Relative Humidity Range (Storage)	20% to 95% non-condensing
Shelving Capacity	EX150 Cart

Platform Maximum Load	55.06 lb (25 kg)
Platform Depth	19.61 in (49.82 cm)
Platform Width	19.29 in (49.00 cm)
Maximum Height Capacity (platform)	15.75 in (40 cm)
Middle Shelf Maximum Load	55.06 lb (25 kg)
Middle Shelf Depth	19.61 in (49.82 cm)
Middle Shelf Width	19.29 in (49.00 cm)
Middle Shelf Height	User- Adjustable
Bottom Shelf Maximum Load	55.06 lb (25 kg)
Bottom Shelf Depth	19.61 in (49.82 cm)
Bottom Shelf Width	19.29 in (49.00 cm)
Bottom Shelf Height	User- Adjustable
Basket Maximum Load	22.02 lb (10 kg)

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